

**Collection of bursa tissue for the detection of
Infectious Bursal Disease Virus (IBDV)
using the rt/pcr/rflp assay**

Procedure for samples originating INSIDE the continental United States:

Collect bursa and cut each bursa in half. Place one half of each bursa into a sterile bag or container. A total of 4 to 5 bursa halves may be pooled from the same flock/collection site. Freeze the bursa.

Place on dry ice or ice packs in an insulated container and ship OVERNIGHT for weekday delivery to:

Dr. Jackwood
Edgington Hall / OARDC
1680 Madison Ave.
Wooster, Ohio 44691

Lab contact:

Susan Sommer
Email: sommer.31@osu.edu
Phone: 330-263-3749
Fax: 330-263-3760

This protocol is for samples originating within the continental United States ONLY.